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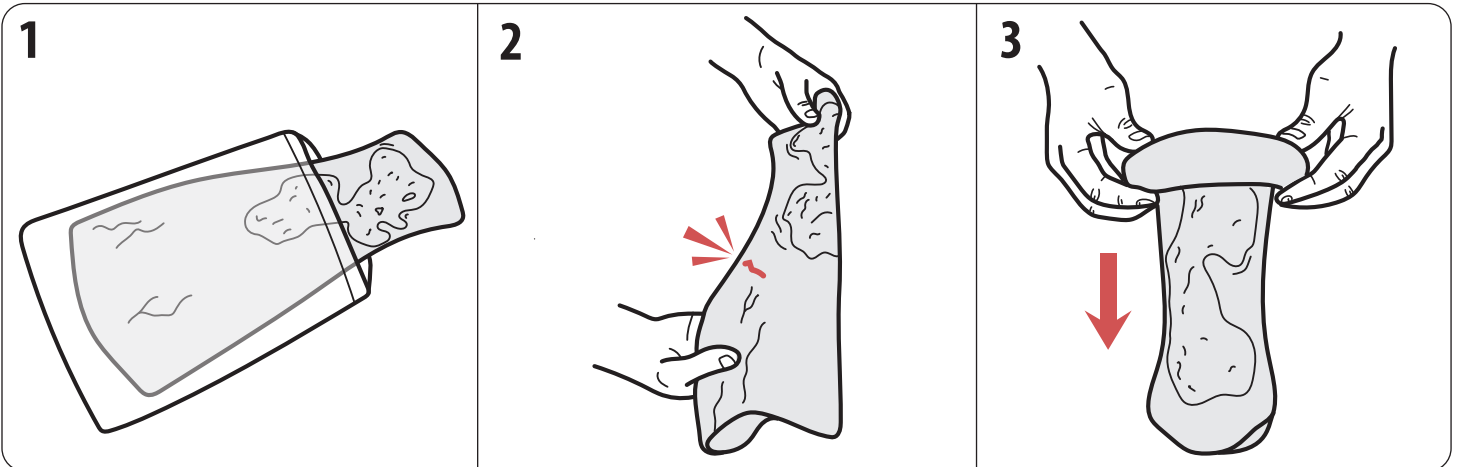
Hinweise

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Ihr MEDICAL FX Germany Team

Herstellerinformationen

MEDICAL FX Germany GmbH - Inh. Nicholas Krützfeldt

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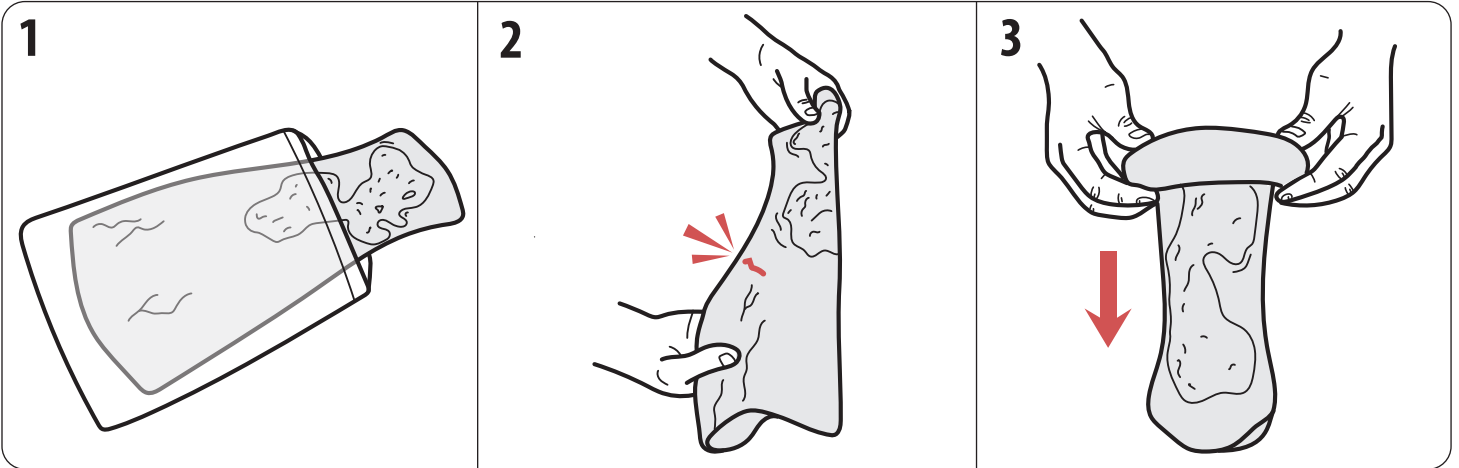
Kundenservice: info@medicalfx.de

SLEEVE INSTRUCTIONS

Important Information

- Read the instructions thoroughly before using the product.
- Improper handling may damage the model.
- Contact with sharp or hot objects may damage the surface of the model.
- Excessive stretching and pulling may damage the sleeve.

Carefully remove the sleeve from the packaging. The sleeve is delivered in a PVC bag. You can use it for further storage, but storage in the cotton bag is sufficient.

Check the model for visible damage. If you notice any damage, please contact our customer service department:

info@medicalfx.com

The easiest way to put on the sleeve is to roll it onto the lower leg. To do this, first roll the sleeve from the top (wider end) to the bottom. This creates a relatively thick, round ring, with the inside of the sleeve forming the outside.

Place the ring opening over the foot of the practice dummy and guide it up to the ankle. Then gently roll the sleeve upward along the lower leg until it is in position.

You can conceal the ends of the sleeve (knees, ankles) with cloths, clothing, or similar items to achieve an even more realistic appearance.

Please do not try to force the sleeve over the leg like a tight stocking. This could damage the sleeve.

To make it easier to put on, you can dust the inside of the sleeve with a little bit of baby powder if necessary.

Maintenance and Storage

Cleaning

- Clean the model as needed with a soft cloth, warm water, and soap.
- Do not use harsh household cleaners that could damage the material.
- You can use disinfectant or rubbing alcohol to remove adhesive and paste residue.
- Allow the sleeve to air dry before storing or reusing it.

Storage

- Store the model in the cotton bag provided.
- Keep the model in a dry and cool place.
- Avoid direct sunlight and extreme temperatures.

Problems and Solutions

Problem	Solution
The sleeve doesn't fit properly	Roll the sleeve down the lower leg of the dummy from the top, then try to roll it up again. Start higher/lower than before, or twist the sleeve accordingly if it did not fit straight on the leg.
Visible damage to the sleeve	Contact customer service for a possible repair or replacement. Please provide photos of the damages.

If you have any further problems or questions, please contact our customer service department at info@medicalfx.de.

Technical Data

Material	High-quality, skin-like silicone
Measurements	Approx. 39 × 16 × 5 cm; Suitable for average sized lower legs
Weight	Approx. 630g
Compatibility	Suitable for use with all common dressing materials and wound care products

We hope this model will be of valuable service to you in your work. If you have any further questions, please do not hesitate to contact us. We are always available to assist you.

Your MEDICAL FX Team Germany

Manufacturer Information

MEDICAL FX Germany GmbH - CEO. Nicholas Krützfeldt

Website: medicalfx.de

Customer Service: info@medicalfx.de